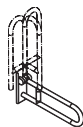
- Ⓓ Montageanleitung
- ⒼⒷ Installation instructions
- Ⓕ Montage
- ⒹⒻ Montage van
- Ⓘ Istruzioni di montaggio
- Ⓔ Montaje instrucciones
- ⒶⒻ Instrukcja montażu

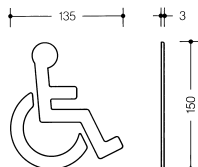
Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
N° de artículo
Numer produktu

4 - 5


Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Advertencias
Ostrzeżenia

6 - 9


Zweckbestimmung
Proper use
Affectation
Toepassingen
Conformità di norma
Usò previsto
Przeznaczenie

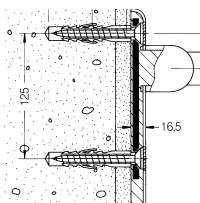10 - 11


Befestigungsarten
Fixing types
Types de fixations
Bevestigingswijzen
Sistemi di fissaggio
Tipos de fijación
Rodzaje mocowań

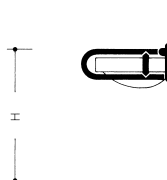12 - 21

Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Indicaciones de montaje
Wskazówka dotycząca montażu

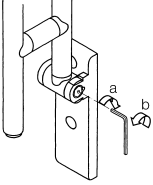22 - 25

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrucciones de montaje
Instrukcje montażu

26 - 39

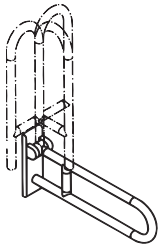
Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Mantenimiento
Konservacja

40 - 43

Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja


44 - 45

801.50.100
801.50.110
801.50.120
801.50.130


801.50.101
801.50.111
801.50.121
801.50.131

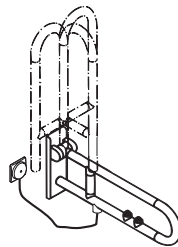CE

801.50.500
801.50.510

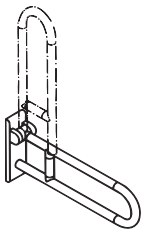
CE

801.50.200
801.50.210
801.50.220


801.50.201
801.50.211
801.50.221

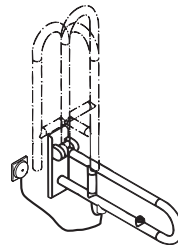CE

801.50.550
801.50.560

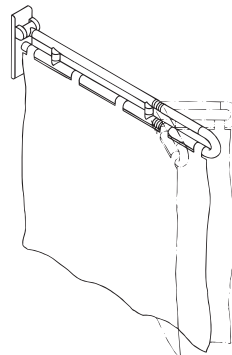
CE

801.52.10001
801.52.10004


801.52.10021
801.52.10023
801.52.10030
801.52.10031
801.52.10032

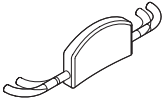801.51.900


CE

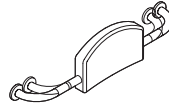801.51.910

CE

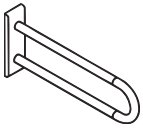
801.50.300
801.50.310
801.50.320

801.50.301
801.50.311
801.50.321


CE

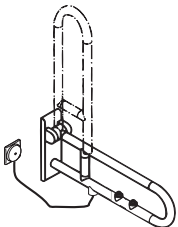801.50.700
801.50.710

CE

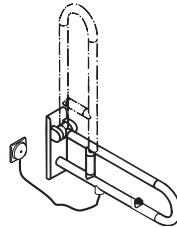801.50.750
801.50.760

CE

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**WARNING!****Use of HEWI Hinged support rail**

The HEWI hinged support rail is designed to withstand a maximum static working load of 100 kg (vertical) and 35 kg (horizontal), the equivalent of 150 kg body weight assuming normal use. Do not use this support for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the support or the joint. In such cases, the support must be examined by the company appointed to carry out maintenance work prior to the support being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI hinged support rail in the manner described in the instructions. Any improper use e.g. as a gymnastic apparatus or climbing aid can cause an accident with persons potentially suffering serious injury.

No persons (particularly children) may be in the area underneath the support when it is being folded down as this can cause an accident with serious injury

Assuming normal use, the support is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the support. The fact that the support bends to a certain extent under load is normal. Should however the support become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

Installation of HEWI Hinged support rail

The smooth and safe functioning of the HEWI hinged support rail largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the support's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the support yourself.

When the licensed contractor mounts the HEWI hinged support rail, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the support and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. The instructions accompanying the wall plugs should be observed in full.

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**Proper use in accordance
with Directive 93/42/EWG**

Aids for barrierfree usage

Stationary supports are generally specified for bathrooms where they tend to be located on both sides of washbasins and toilets. They can however be used elsewhere if the need arises.

They provide users with support and something to hold on to, thus preventing them from accidentally falling off the toilet. Furthermore, persons with a physical disability can use them to help themselves stand up, enabling them to make use of bathroom facilities without help.

The toilet paper holder integrated into the front part of the support, depending on the type, enables the user to remove toilet paper with one hand and can be mounted in a left hand or right hand position or, if required, on both sides of the toilet.

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
It is absolutely necessary to consider the appropriate assembly instructions and further referings on page 12 - 13.

Serie 801

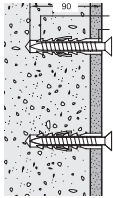Suitable for fixing to walls of:

Plastic wall plug M10
 Countersunk bolt: 10 x 95

- Beton (>B15)

**Stationary support rail
 Hinged support rail
 Shower spray guard**

Hinged seat

Back rest
 suitable for fixing on Support rail

BM10.2

Supplied with:
 2 bolts,
 2 wall plugs

BM20.4

Supplied with:
 4 bolts
 4 wall plugs

BM10.2.1

Supplied with:
 2 bolts
 2 centering sleeves
 2 wall plugs

- solid bricks of
 lightweight
 concrete
 - aircrete
 - sand-lime
 perforated bricks

BM11.2


Supplied with:
 2 bolts
 2 basket sleeve
 2 anchor sleeve

BM21.4

Supplied with:
 4 bolts
 4 basket sleeve
 4 anchor sleeve

BM11.2.1

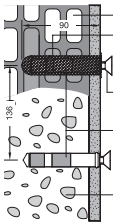
Supplied with:
 2 bolts
 2 centering sleeves

Perforated brick
 Basket sleeve
 HT-SC 22 x 85
 Countersunk bolt
 M 10 x 30 DIN 7991
 Anchor sleeve
 HT-AS M10
 Solid brick

With basket sleeve
 - vertically-perforated
 bricks
 - hollow blocks
 - sand-lime
 perforated bricks
 - other
 perforated bricks

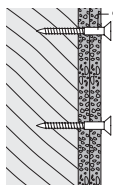Without basket sleeve
 - concrete
 - solid bricks
 - solid pumice and
 other solid materials
 - natural stone
 - sand-lime
 solid bricks

Countersunk wood screw
 M10 x 90

Suitable for fixings to light-weight partition walls with support frame of hardwood min. 20 mm thick.

BM12.2


Supplied with:
 2 bolts

BM22.4

Supplied with:
 4 bolts,

BM12.2.1

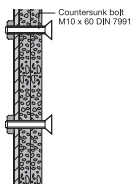Supplied with:
 2 stainless steel locknut
 2 centering sleeves

Suitable for fixings to light-weight partition walls with integrated steel plates.

BM13.2

Supplied with:
 2 bolts

BM23.4

Supplied with:
 4 bolts

BM13.2.1

Supplied with:
 2 bolts
 2 centering sleeves

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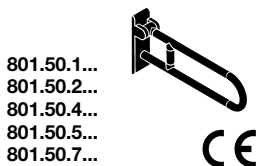
a: siehe Seite 22

(GB) Position template and stick to the wall. Take into consideration the necessary free space $L = 160$ mm, min. between centre line and wall.
 a = Installation height between finished floor and centre of upper support rail (1).

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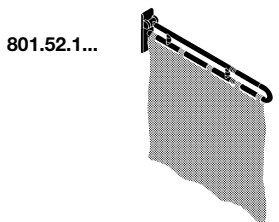
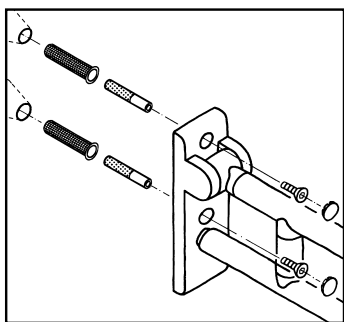
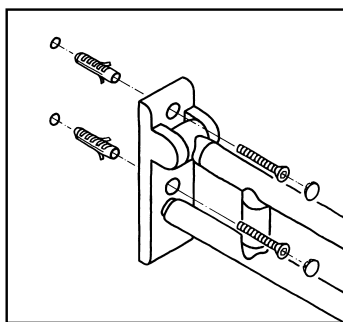
Fixing type BM10.2

Test structure of wall! Drill holes 14 mm \varnothing and remove template. Drive in plastic plugs and tighten support. Snap in caps.

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Fixing type BM11.2

Test structure of wall! Drill holes for plugs 22 mm \varnothing , remove template. Install injection anchor and allow cement to set as per instruction (injection mortar not supplied from HEWI).

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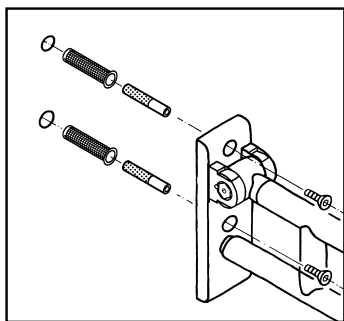
Remove protective sleeves and tighten support (protect screws against corrosion with grease). Snap on caps.

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Fixing type BM11.2

Test structure of wall! Perforated brick: Drill plug holes 22 mm \varnothing x 95 mm deep.

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Remove drill template. Insert filter sleeve and inject HIT-HY70 injection mortar from cartridge (from local supplier; observation of manufacturer's instructions vital). Insert anchor sleeve.

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Maintenance of hinged support rail

The HEWI hinged support rail is not maintenance free! The correct functioning of the brake device that prevents the support from dropping accidentally can be impaired by external influences (temperature, moisture). The support ought to be inspected by the maintenance company at least once a month to ensure its correct functioning. The persons entrusted with the maintenance work must be familiar with the functionality of the support and the correct setting of the brake device. The appropriate instructions and description are enclosed with every support. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. Do not carry out the maintenance work yourself as this could lead to the correct functioning of the support being severely impaired with the risk of persons potentially suffering severe injuries.

The maintenance work must include examination of the brake device to ensure the support folds down slowly and smoothly. If necessary, the friction device should be re-adjusted. Observe these instructions. Under no circumstances should the brake discs or joint be greased or lubricated. Moreover, the wall fixing must be inspected to ensure everything is as it should be. Non-compliance with these instructions can cause an accident with persons potentially suffering serious injury.

Damage in transit

The support left the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the support functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the support is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the support in its packaging whenever possible. Once the support has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's hinged support rail, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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
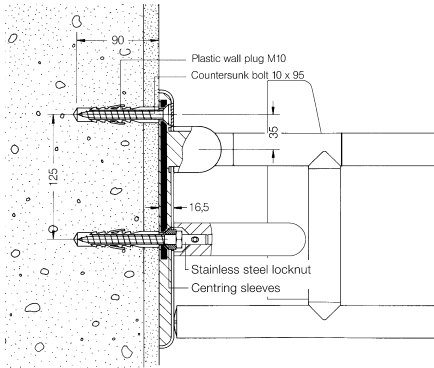
BM10.2.1

For fitting to supports with lift-up feature or stationary supports with wall mounting plate with fixing BM10.2:

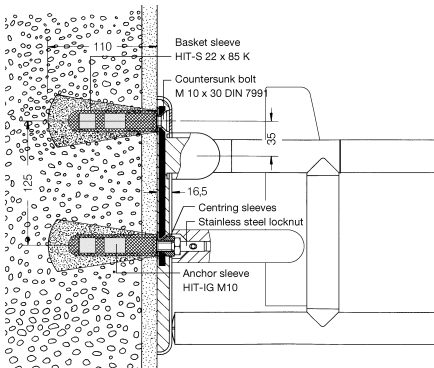
Supplied with 2 bolts (inox) and
 2 stainless steel locknut
 2 centring sleeves


BM11.2.1

For fitting to supports with lift-up feature or stationary supports with wall mounting plate with fixing BM11.2:

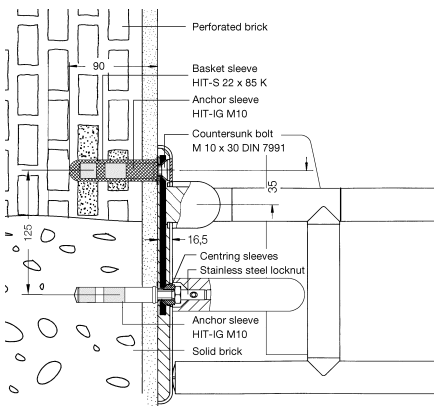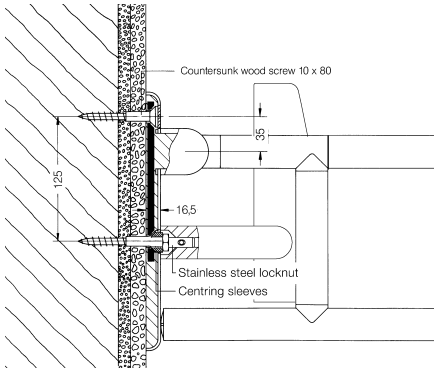Supplied with 2 bolts (inox) and
 2 stainless steel locknut
 2 centring sleeves


BM12.2.1

For fitting to supports with lift-up feature or stationary supports with wall mounting plate with fixing BM12.2:

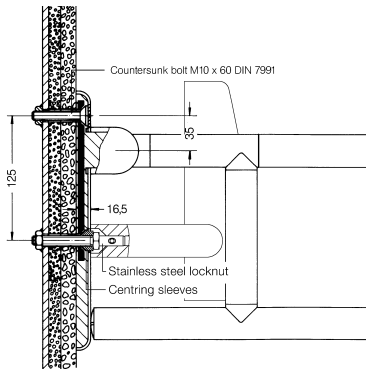
Supplied with 2 bolts (inox) and
 2 stainless steel locknut
 2 centring sleeves

BM13.2.1

For fitting to supports with lift-up feature or stationary supports with wall mounting plate with fixing 801.13:

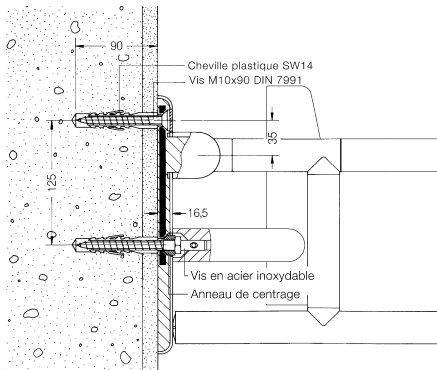Supplied with 2 bolts (inox) and
 2 stainless steel locknut
 2 centring sleeves

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Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information. Only use detergents with a ph-rating will do between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

**For further information please contact
HEWI Technical Support, tel. +49 5691 82-300.**

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HEWI Heinrich Wilke GmbH