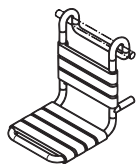
- Ⓓ Montageanleitung
- ⒼⒷ Installation instructions
- Ⓕ Montage
- ⒼⓃ Montage van
- Ⓘ Istruzioni di montaggio
- Ⓔ Montaje instrucciones
- Ⓐ Instrukcja montażu

Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
N° de artículo
Numer produktu

4

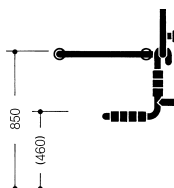
Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Advertencias
Ostrzeżenia

5 - 8


Zweckbestimmung
Proper use
Affectation
Toepassingen
Conformità di norma
Usò previsto
Przeznaczenie

9

Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Indicaciones de montaje
Wskazówka dotycząca montażu

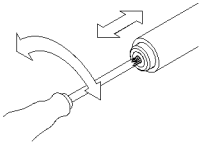10 - 11

Gebrauchsanleitung
Instructions for use
Mode d'emploi
Gebruiksaanwijzing
Istruzioni d'uso
Modo de empleo
Instrukcja użytkowania

12

Hinweise
Tips
Conseil
Instructies
Istruzioni
Indicaciones
Wskazówki

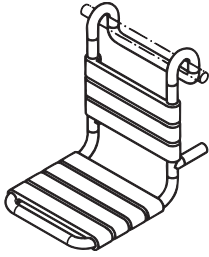
13 - 14






Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja

15 - 18

801.51.100

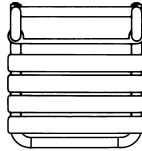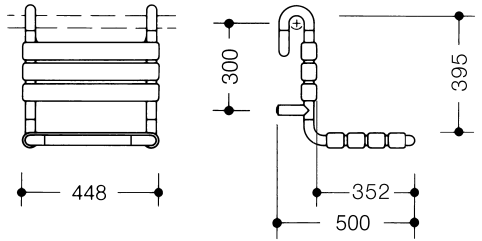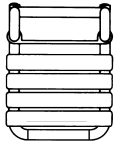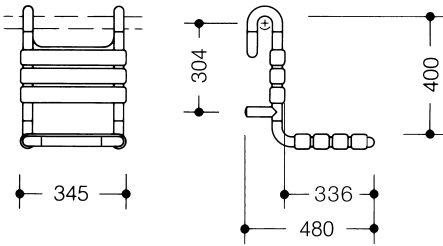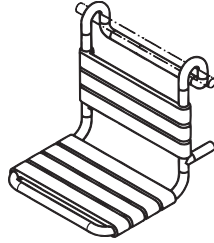801.51.110

Warnhinweise!

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WARNING!

Use of HEWI removable hanging

The HEWI seat is designed to withstand a maximum static working load of 150 kg (vertical). Do not use this seat for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the seat. In such cases, the seat must be examined by a specialised company work prior to the seat being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI seat in the manner described in the instructions. Any improper use e.g. as a climbing aid can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the seat is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the seat. The fact that the seat bends to a certain extent under load is normal. Should however the seat become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

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GB

Intended use

The HEWI removable hanging seat with lift-up feature is designed for the use in bathrooms, especially in showers. It provides people, particularly patients or persons with a handicap, a seat while they are taking a shower.

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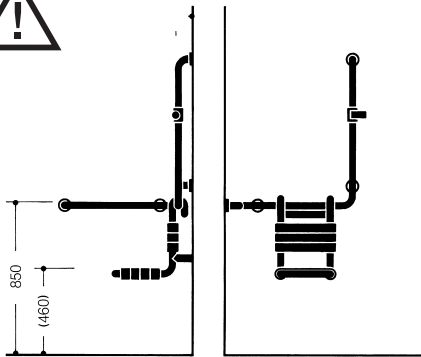
GB The removable bath seat can be used in combination with the following products:

HEWI support rail	801.36.130
	801.36.1S
HEWI shower rail	801.35.100
	801.35.110
	801.35.1S
	801.35.200
	801.35.2S
	801.35.210
	801.35.21S
	801.35.300
	801.35.3S
	801.35.400
	801.35.4S
HEWI L-shaped support rail	801.33.2S

From a length of 1039 mm on, an additional wall fixing is necessary because the distance between the supports may not exceed 850 mm (horizontal rail).

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801.51.100
801.51.110

(D) **HEWI Einhängesitze**
Wandpuffer (1).

(GB) **HEWI removable hanging seats**
Wall buffer (1).

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To set the angle of the seat, turn the wallbuffer in or out with a Torx-screwdriver T40. Max. adjustment travel 20 mm.

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GB**Damage in transit**

The seat left the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the seat functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible. Once the seat has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's seat with lift-up feature, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough.

If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a pH-rating will do between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling.

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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HEWI Heinrich Wilke GmbH