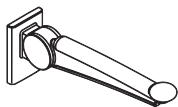
- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- ⓃⓁ **Montage van**
- Ⓘ **Istruzioni di montaggio**
- Ⓔ **Montaje instrucciones**
- Ⓟ **Instrukcja montażu**

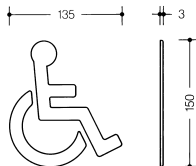
Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
N° de artículo
Numer produktu

4 - 5


Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Advertencias
Ostrzeżenia

6 - 9


Zweckbestimmung
Proper use
Affectation
Toepassingen
Conformità di norma
Usò previsto
Przeznaczenie

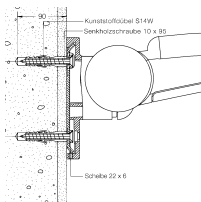10


Befestigungsarten
Fixing types
Types de fixations
Bevestigingswijzen
Sistemi di fissaggio
Tipos de fijación
Rodzaje mocowań

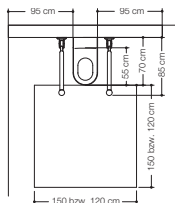11 - 18


Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Indicaciones de montaje
Wskazówka dotycząca montażu

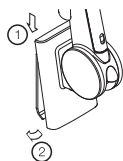20 - 29

Montageanleitungen
Installation instructions
Notice de montage
Montage aanwijzing
Istruzioni di montaggio
Instrucciones de montaje
Instrukcje montażu

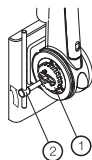30 - 41

Gebrauchsanleitung
Instructions for use
Mode d'emploi
Gebrauksaanwijzing
Istruzioni d'uso
Modo de empleo
Instrukcja użytkowania

42 - 55

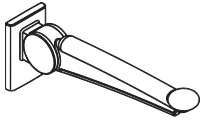
Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Mantenimiento
Konserwacja

56 - 59


Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja

60 - 63

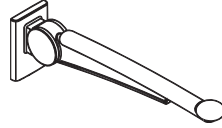
802.50.0165
802.50.0170
802.50.0175
802.50.W0165
802.50.W0170
802.50.W0175


802.50.0180
802.50.0185
802.50.0190
802.50.W0180
802.50.W0185
802.50.W0190

CE

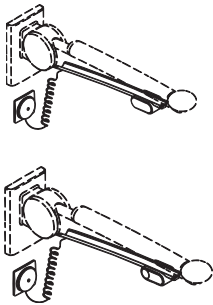CE

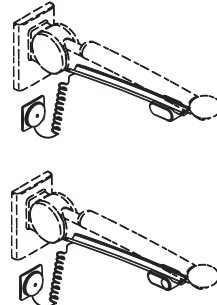
802.50.020R
802.50.020L
802.50.W020R
802.50.W020L


802.50.030R
802.50.030L
802.50.W030R
802.50.W030L

CE

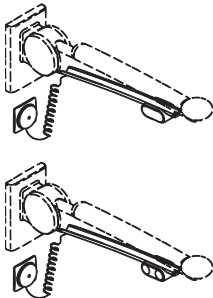CE

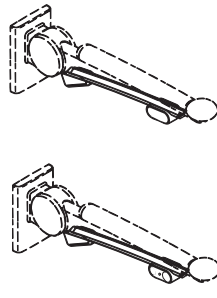
802.50.040R
802.50.040L
802.50.W040R
802.50.W040L

802.50.050R
802.50.050L
802.50.W050R
802.50.W050L

CE

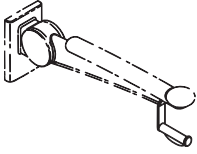CE


802.50.010
802.50.W010


CE

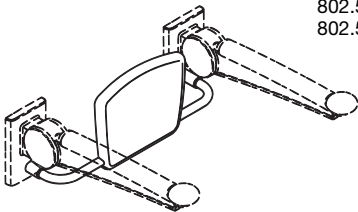802.50.015
802.50.W015

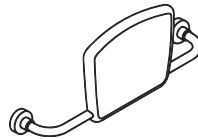802.50.06092
802.50.06097

802.50.07092
802.50.07097

Warnhinweise!

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WARNING!

Use of HEWI support with lift-up feature

The HEWI support with lift-up feature is designed to withstand a maximum static working load of 100 kg (vertical) and 40 kg (horizontal), the equivalent of 150 kg body weight assuming normal use. Do not use this support for exceeding weights. Any non-compliance with loading limits can cause permanent damage to the support or the joint. In such cases, the support must be examined by the company appointed to carry out maintenance work prior to the support being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI support with lift-up feature as described in the instructions. Any improper use, e.g. as a gymnastic apparatus or climbing aid, can cause an accident with persons potentially suffering serious injury.

No persons (particularly children) may be in the area underneath the support when it is being folded down, as this can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the support is of a flexible nature thanks to the materials used in its manufacture. Such flexibility reduces the risk of injury if anyone bumps into the support. The fact that the support bends to a certain extent under load is normal. Should, however, the support become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

Installation of HEWI support with lift-up feature

The smooth and safe functioning of the HEWI support with lift-up feature largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the support's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the support yourself.

When the licensed contractor mounts the HEWI support with lift-up feature, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the support and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. The instructions accompanying the wall plugs should be observed in full.

Once installation has been completed, you must carry out a functionality check!

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
It is absolutely necessary to consider the appropriate assembly instructions and further referrals on page 11!

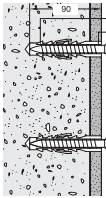LifeSystem
 Suitable for fixing to walls of:

Plastic wall plug M10
 Countersunk bolt 10 x 95

- concrete

Hinged support rail

Hinged seat

Back rest
 with roses

BM10.3

BM10.3

BM40.6

Supplied with:
 3 bolts,
 3 washers
 3 wall plugs

Supplied with:
 3 bolts
 3 washers
 3 wall plugs

Supplied with:
 2 bolts
 2 washers
 2 stainless steel locknut

BM11.3

BM11.3

BM41.6

- solid bricks of light weight concrete
 - aircrete
 - sand-lime perforated bricks

Supplied with:
 3 bolts
 3 washers
 3 basket sleeve
 3 anchor sleeve

Supplied with:
 3 bolts
 3 washers
 3 basket sleeve
 3 anchor sleeve

Supplied with:
 2 bolts
 2 washers
 2 basket sleeve
 2 anchor sleeve

With basket sleeve

- vertically-perforated bricks
 - hollow blocks
 - sand-lime perforated bricks
 - other perforated bricks

Supplied with:
 3 bolts
 3 washers
 3 basket sleeve
 3 anchor sleeve

Supplied with:
 3 bolts
 3 washers
 3 basket sleeve
 3 anchor sleeve


Suitable for fixing on wall plates:

BM30.2


Supplied with:
 2 bolts
 2 washers
 2 stainless steel locknut

Without basket sleeve

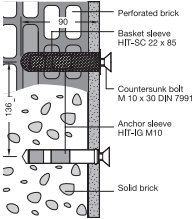- concrete
 - solid bricks
 - solid pumice and other solid materials
 - natural stone
 - sand-lime solid bricks

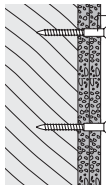Perforated brick
 Basket sleeve HIF-SC 22 x 85
 Countersunk bolt M 10 x 30 DIN 7991
 Anchor sleeve HIF-IG M10
 Solid brick

Countersunk wood screw 10 x 80

Suitable for fixings to light-weight partition walls with support frame of hardwood min. 20 mm thick.

BM12.3


BM12.3

BM42.6

Supplied with:
 3 bolts
 3 washers

Supplied with:
 3 bolts,
 3 washers

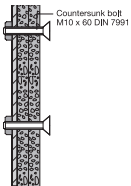Supplied with:
 2 bolts
 2 washers

Countersunk bolt M10 x 60 DIN 7991

Suitable for fixings to light-weight partition walls with integrated steel plates.

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
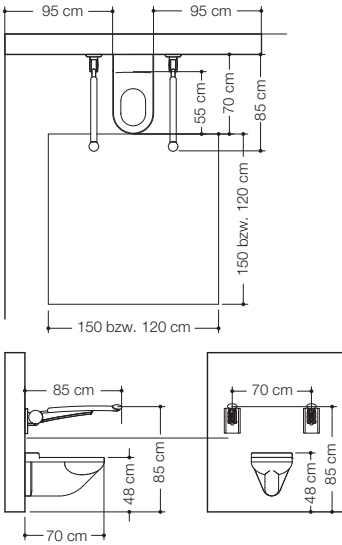
Toilet area


	private DIN 18 025 part 1	private DIN 18 025 part 2	private DIN 18 024 part 3
Fixing height including seat	48 cm	individual	48 cm
WC-depth (distance between lower edge and wall)	70 cm	individual	70 cm
Sideway area of movement	95 cm left or rechts	individual	95 cm and right
Area of movement in front of the WC	150 cm deep 150 cm wide	120 cm deep 120 cm wide	150 cm deep 150 cm wide
Sideway distance from the WC to the wall	30 cm	individual	95 cm
Supports with lift-up feature	individual	individual	both sides 85 cm high
Fixing distance with integrated flush with integrated toilet roll holder	individual individual individual	individual individual individual	70 cm left and right left and right
Backrest rail	individual	individual	55 cm behind the front edge

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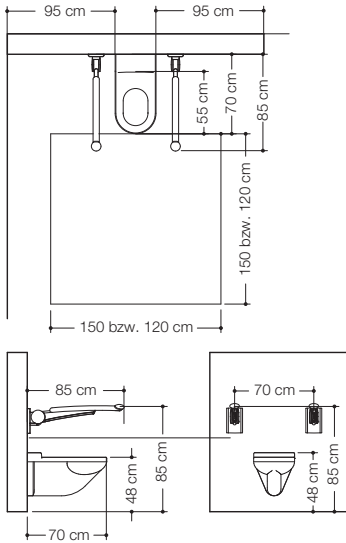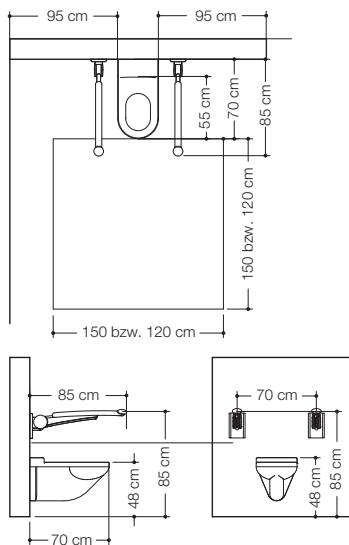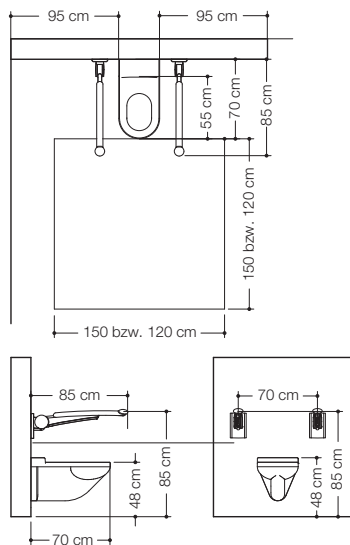
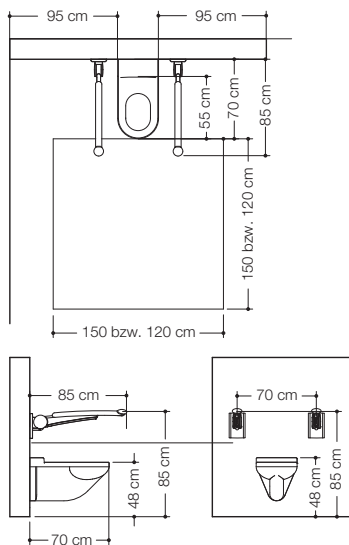
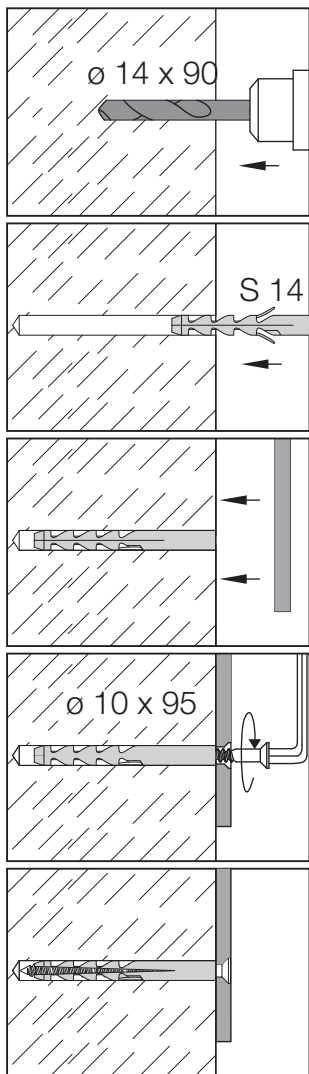
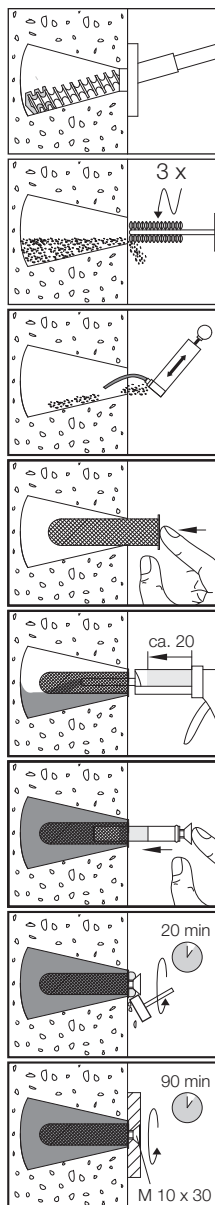
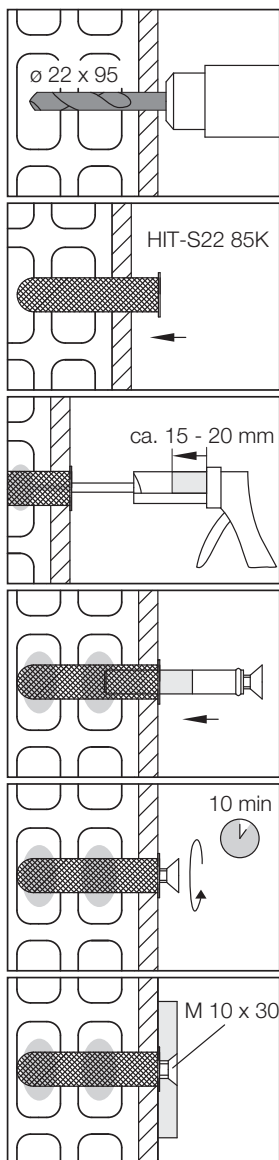
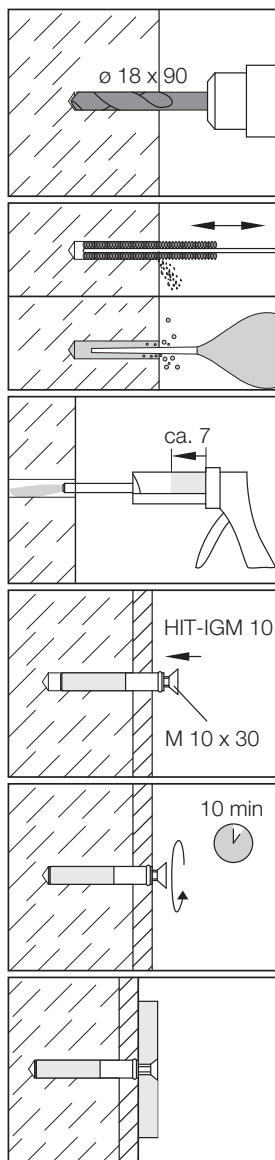
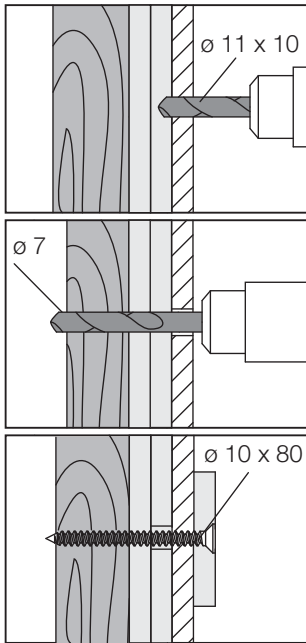
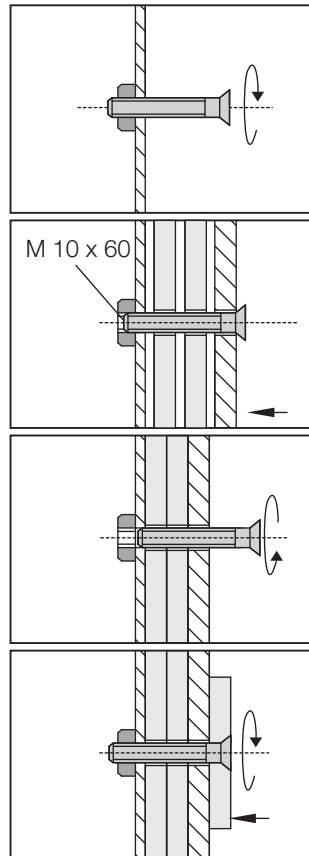
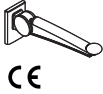
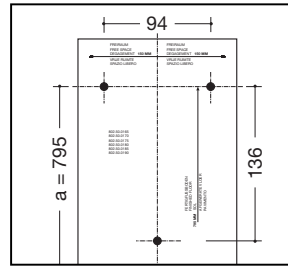
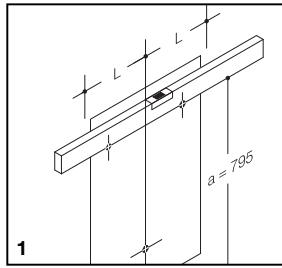
Bohrschablone.

(GB) Position template and stick to the wall. Take into consideration the necessary free space $L = 150$ mm, min. between centre line and wall.
 $a =$ Installation height from finished floor.

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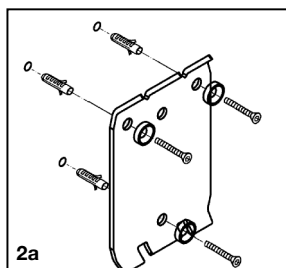
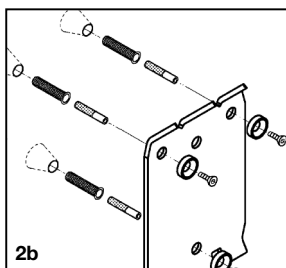
Fixing type BM10.3

Test structure of wall! Drill holes 14 mm ϕ and remove template. Drive in plastic plugs and tighten wallplate.

Fixing type BM11.3

Test structure of wall! Drill holes for plugs 22 mm ϕ , remove template. Install injection anchor and allow cement to set as per instruction (injection mortar not supplied from HEWI). Remove protective sleeves and tighten

wallplate (protect screws against corrosion with grease).

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Fixing type BM11.3

Test structure of wall! Solid brick: Drill plug holes 18 mm ϕ x 90 mm deep. Remove drill template. Ensure plug holes are clean and free of dust.

Apply HIT-HY70 injection mortar or similar from cartridge (from local supplier). Observation of manufacturer's instructions vital. Insert anchor sleeve.

Important! Protect M 10 internal thread. Observe manufacturer's recommended setting time and tighten wallplate (protect screws against corrosion with grease).

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Turn socket spanner in clockwise direction as far as it will go.

Use spanner (2) to pull setting element (1) out of bearing (3) as far as it will go. **Caution – risk of injury!** Once setting element has been disengaged, the force of the spring acts directly on spanner!

Turn setting element (1) with socket spanner (2) until the required spring tension has been reached. *To increase spring tension: turn spanner in anticlockwise direction (as illustrated). To reduce spring tension: turn spanner in clockwise direction.*

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GB

Maintenance of support rail with lift-up feature

The HEWI support with lift-up feature is not maintenance-free! The braking effect of the spring when folding down the support can change, e.g. due to the support having been adjusted or to external influences.

The support rail should be examined by maintenance staff, at least every month, to ensure it functions perfectly. The maintenance staff concerned must be familiar with the functionality of the support and the adjustment of the spring. Appropriate instructions and descriptions are enclosed with every support rail. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. Do not carry out maintenance work yourself, as serious impairment to the support's proper functioning can occur, potentially resulting in accidents with persons suffering serious injury.

In the course of the maintenance work, the braking effect of the spring must be checked, as must the slow, smooth lowering movement of the support. The spring setting should be reset if necessary. Comply with the attached instructions whilst proceeding. Under no circumstances should the bearing be greased or oiled. A check should also be made to ensure that the support is fastened firmly to the wall. Non-compliance with this instruction can result in accidents with persons potentially suffering serious injury.

Damage in transit

The support leaves the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the support functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the support is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the support in its packaging whenever possible. Once the support has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's support with lift-up feature, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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GB

Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information. Only use detergents with a pH-rating of between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used. Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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HEWI Heinrich Wilke GmbH