- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- ⓃⓁ **Montage van**
- Ⓘ **Istruzioni di montaggio**
- Ⓟ **Instrukcja montażu**

Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
Numer produktu

4

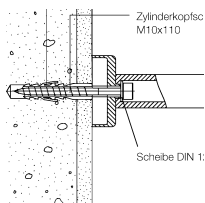
Zweckbestimmung
Proper use
Affectation
Toepassingen
Conformità di norma
Przeznaczenie

5


Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Ostrzeżenia

6 - 11


Befestigungsarten
Fixing types
Types de fixations
Bevestigingswijzen
Sistemi di fissaggio
Rodzaje mocowań

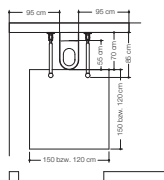12 - 19

Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Wskazówka dotycząca montażu

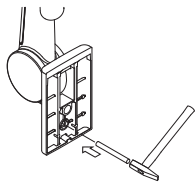20 - 28

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrukcje montażu

30 - 45

Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Konservacja

46 - 48

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WARNING!

Use of HEWI backrest rail

The HEWI backrest rail is designed to withstand a maximum static working load of 50 kg (horizontal). Do not use this support for exceeding loads. Any non-compliance with loading limits can cause permanent damage to the rail. In such cases, the rail must be examined by the company appointed to carry out maintenance work prior to the support being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI backrest rail as described in the instructions. Any improper use, e.g. as a seat or climbing aid, can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the backrest rail is of a flexible nature, thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the rail. The fact that the rail bends to a certain extent under load is normal. Should however the rail become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

Product compatibility

The HEWI backrest rail may only be used in conjunction – with HEWI supports with lift-up feature (Item No. 802.50...) from the LifeSystem range or as independent product with rose fittings (item No. 802.50.070...). HEWI accepts no liability whatsoever for damage to persons or property that occurs as a result of the rail being or having been used in conjunction with other products (made by HEWI or any other manufacturer).

Installation of the HEWI backrest rail

The smooth and safe functioning of the HEWI backrest rail largely depends on its correct fixing to the given wall or support with lift-up feature, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the rail's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. Therefore, we advise you not to install the support yourself.

When the licensed contractor mounts the HEWI backrest rail, he should ensure that the entire area of the wall to which the rose is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the rail and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned.

If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

The instructions accompanying the wall plugs should be observed in full.

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

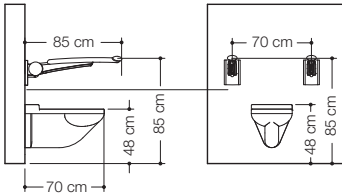
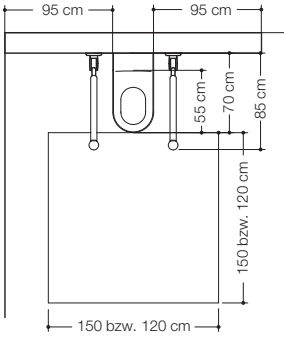
Toilet area


	private DIN 18 025 part 1	private DIN 18 025 part 2	private DIN 18 024 part 3
Fixing height including seat	48 cm	individual	48 cm
WC-depth (distance between lower edge and wall)	70 cm	individual	70 cm
Sideway area of movement	95 cm left or right	individual	95 cm and right
Area of movement in front of the WC	150 cm deep 150 cm wide	120 cm deep 120 cm wide	150 cm deep 150 cm wide
Sideway distance from the WC to the wall	30 cm	individual	95 cm
Supports with lift-up feature	individual	individual	both sides 85 cm high
Fixing distance with integrated flush	individual	individual	70 cm
with integrated toilet roll holder	individual	individual	left and right left and right
Back rest rail	individual	individual	55 cm behind the front edge

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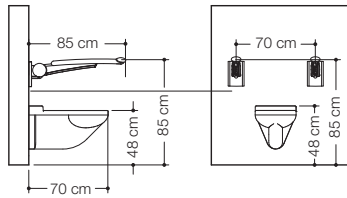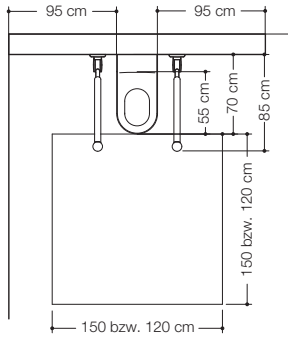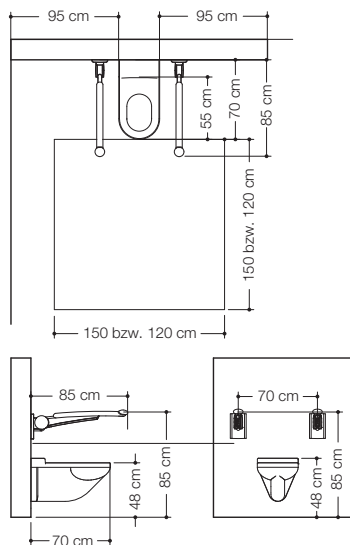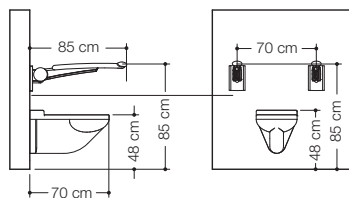
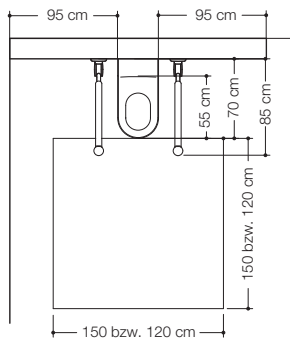
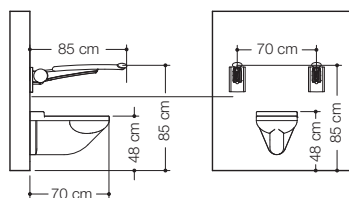
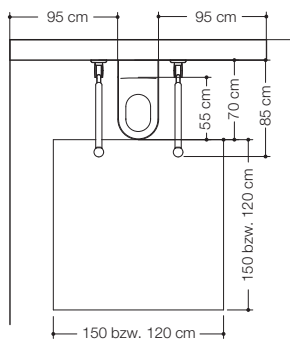
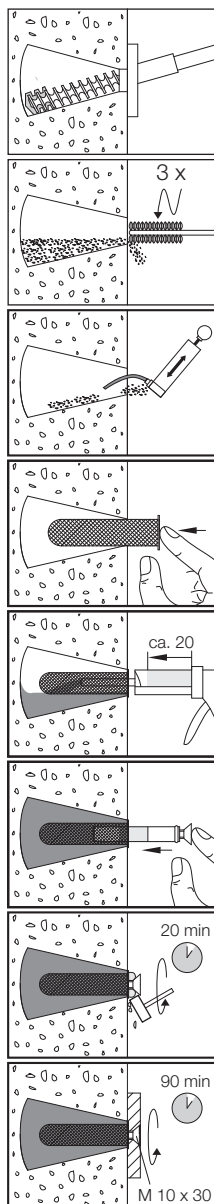
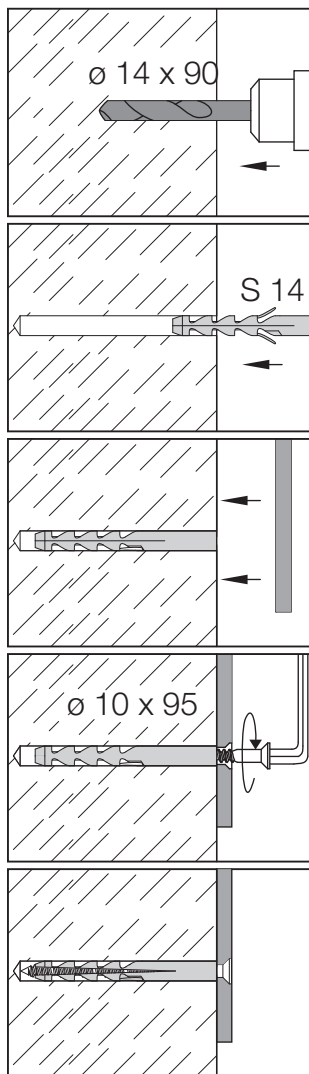
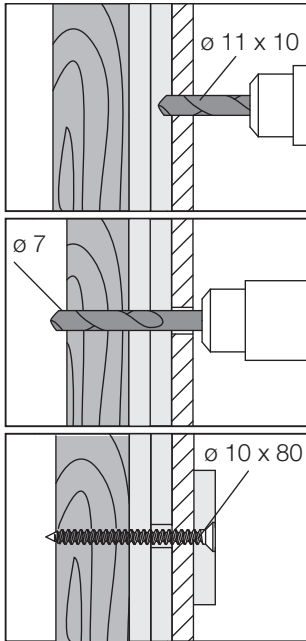
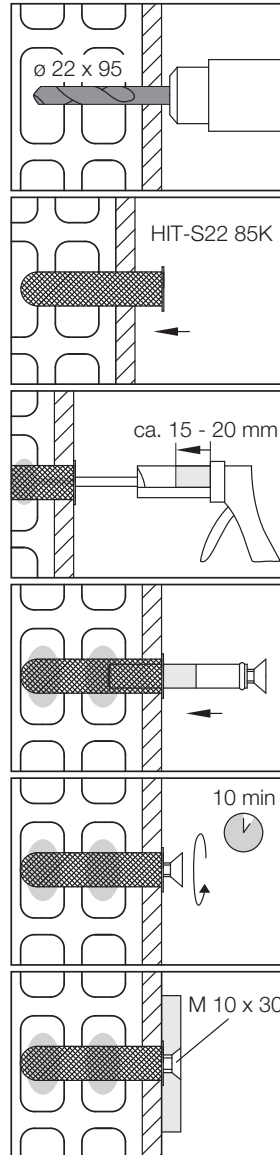
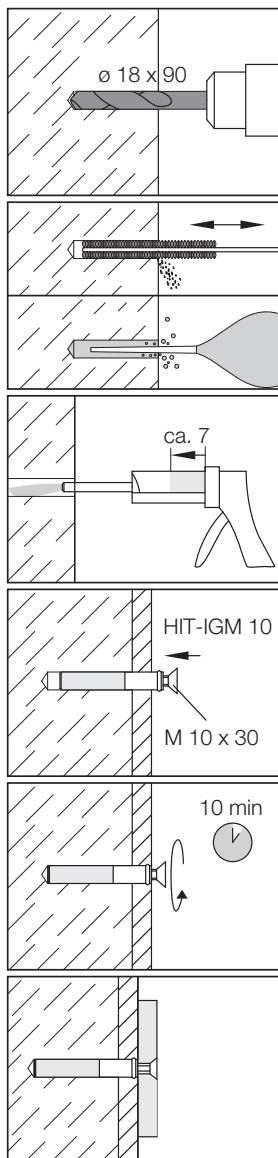
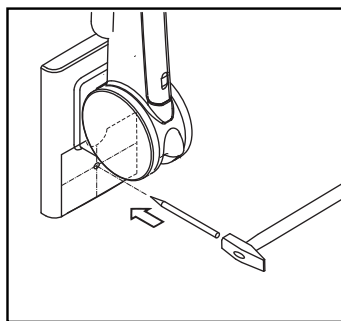
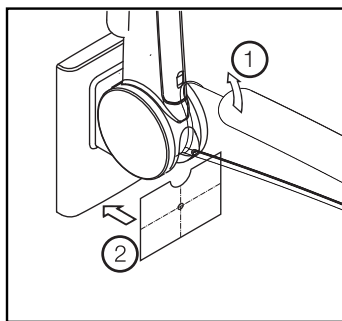
Mounting backrest rail
(support rail on both sides)

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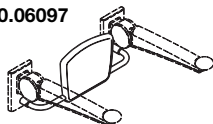
2. Bohrloch ankörnen.

GB 1. Fold support rail up (1) and
stick template (2).

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Remove protective sleeves and tighten tubular housing (protect screws against corrosion with grease).

3. Fixing type BM42.6

Test structure of wall! Drill holes for screws 7 mm ø (if mounted on tiles, drill 11 mm ø clearance hole through tiles). Tighten tubular housing.

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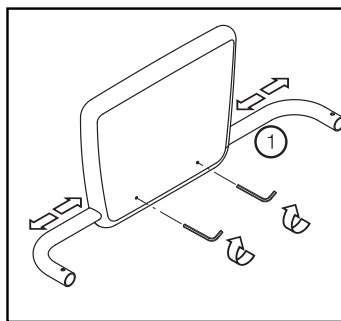
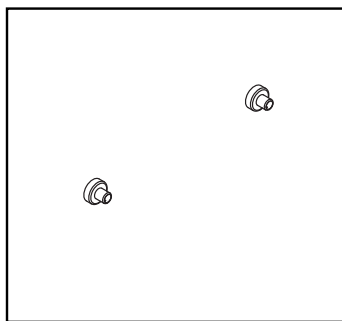
Mounting backrest rail
 (rose fitting on both sides)

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GB 1. Align rose fitting and tubular housing (short or long) and fasten to wall (see Installation Instructions – Rose Fitting page 48). Repeat step 1 for the other rose fitting.

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2. Set mounting tubes to the correct distance and screw firmly in place.

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Maintenance of backrest

The HEWI backrest rail is not maintenance-free!

The backrest rail should be examined by maintenance staff, at least every month, to ensure that the backrest is fastened firmly to the wall or to the support rail. If in doubt, contract HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Non-compliance with this instruction can result in accidents with persons potentially suffering serious injury.

Damage in transit

The backrest leaves the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the backrest functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the backrest is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the backrest in its packaging whenever possible. Once the backrest has been installed, put the protective foil back around it until the backrest needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's backrest, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a pH-rating of between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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HEWI Heinrich Wilke GmbH