- (D) Montageanleitung**
- (GB) Installation instructions**
- (F) Montage**
- (NL) Montage van**
- (I) Istruzioni di montaggio**
- (E) Montaje instrucciones**
- (PL) Instrukcja montażu**

Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
Nº de artículo
Numer produktu

4


Zweckbestimmung
Proper use
Affection
Toepassingen
Conformità di norma
Uso previsto
Przeznaczenie

5

Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Advertencias
Ostrzeżenia

6 - 9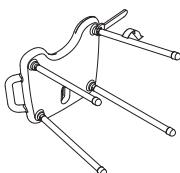

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrucciones de montaje
Instrukcje montażu

10

Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Mantenimiento
Konserwacja

11 - 15

Pflegehinweise
Care tips
Conseils d'entretien
Onderhoudsinstructies
Consigli per la pulizia
Indicaciones para el mantenimiento
Pielęgnacja

16 - 19

Duschhocker
802.51.30092 anthrazitgrau
802.51.30097 lichtgrau

Shower stool
802.51.30092 anthracite grey
802.51.30097 light grey

Tabouret de douche
802.51.30092 gris anthracite
802.51.30097 gris clair

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WARNING!

Use of HEWI shower stool

The HEWI shower stool is designed to withstand a maximum static working load of 150 kg. Do not use this stool for exceeding loads! Any non-compliance with loading limits can cause permanent damage to the stool. In such cases, the stool must be examined by the company appointed to carry out maintenance work prior to the shelf being used again. If in doubt, contact HEWI's after-sales service team on tel. +49 5691 82-300 for further assistance.

Only use the HEWI shower stool as described in the instructions. Any improper use, e.g. as a gymnastic apparatus or rocking with it, can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the shower stool is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into it. The fact that the stool bends to a certain extent under load is normal. Should the stool become permanently deformed, however, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service on tel. +49 5691 82-300.

Assembly of HEWI shower stool

Assembly of the HEWI shower stool should only be entrusted to licensed contractors and carried out in full compliance with the assembly instructions.

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Maintenance of shower stool

In purchasing the HEWI shower stool, you have chosen a high-quality product. Once assembled, the stool requires no special maintenance.

If any problems occur, you should immediately contact HEWI's after-sales service team on tel. +49 5691 82-300 for further assistance.

Damage in transit

The shower stool leaves the factory in perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the seat functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible. Once the seat has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's seat with lift-up feature, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough.

If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a ph-rating of between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used. Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

For further information please contact HEWI, tel. +49 5691 82-300.

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HEWI Heinrich Wilke GmbH